



InnovaPrint Wash

General Purpose 3D Print Resin Cleaner

Harvest Dental Products, LLC
905 Columbia St. Brea, CA 92821 USA
Tel: 1+714-674-7400
Fax: 1+714-674-7402
Email: hello@harvestdental.com
Web: www.harvestdental.com

INSTRUCTIONS FOR USE – English

INTENDED USE

InovaPrint wash is chemically engineered to safely and effectively remove any uncured photopolymer 3D printing resin in the range of 385-420 from SLA, DLP or LCD printed objects. InovaPrint wash is made of organic solvents and is safer and more effective than Isopropyl Alcohol (IPA) and can be used a minimum of 3 times longer than IPA.

WORKING INSTRUCTIONS

- Place the resin print in a tub filled with InovaPrint wash,
- Clean for 10 minutes if using without ultrasonic cleaner or 5 minutes with.
- After cleaning allow the printed object to air dry or rinse with tap water...clean and simple

GUARANTEE

Application guidelines are only a recommendation. We reserve the right to alter the products in terms of design or composition.

INSTRUCCIONES DE USO - Español

USO PREVISTO

El lavado InovaPrint está diseñado químicamente para eliminar de forma segura y efectiva cualquier resina de impresión 3D de fotopolímero sin curar en el rango de 385-420 de objetos impresos con SLA, DLP o LCD. El lavado InovaPrint está hecho de solventes orgánicos y es más seguro y más efectivo que el alcohol isopropílico (IPA) y puede usarse un mínimo de 3 veces más que el IPA.

INSTRUCCIONES DE TRABAJO

- Coloque la impresión de resina en una tina llena con lavado InovaPrint,
- Limpiar durante 10 minutos si se usa sin limpiador ultrasónico o 5 minutos con.
- Después de la limpieza, deje que el objeto impreso se seque al aire o enjuague con agua del grifo ... limpio y simple

GARANTÍA

Las pautas de aplicación son solo una recomendación. Nos reservamos el derecho de modificar los productos en términos de diseño o composición.