# Harvest Temp Fix™

TEMP, TOOTH SHADES    TEMP, ZAHNFARBEN    TEMP, TONALIDADES DEL DIENTE    TEMP, COULEURS DES DENTS

Harvest Dental Products, LLC  
905 Columbia St.  
Brea, CA 92821 USA  
Tel: 1+714-674-7400  
Fax: 1+714-674-7402  
Em: [hello@harvestdental.com](mailto:hello@harvestdental.com)  
Web: [www.harvestdental.com](http://www.harvestdental.com)

<b>INSTRUCTIONS FOR USE – ENGLISH</b>
<b>INTENDED USE</b> Harvest Temp Fix™ tooth shaded acrylic blanks are used to produce resin crowns, bridges and frameworks for long and short term temporary restorations in the Anterior and Posterior regions.
<b>RESISTANCE</b> <b>E-MODULE:</b> > 2.200 MPa <b>BENDING STRENGTH:</b> > 100 MPa
<b>COMPOSITION</b> _ Polymethyl methacrylate (PMMA) and ester-based cross-linked polymers. _ Color pigments.
<b>SHADE</b> BL, A1, A2, A3, B1, B2
<b>HAZARD WARNINGS:</b> Dust is created during milling of TEMP FIX blanks that can cause irritation of the eyes, skin and airways. Therefore always ensure that the extractor on your milling machine is functioning perfectly.
<b>STORAGE</b> No special measured required.
<b>WORKING INSTRUCTIONS</b> _ Mill resin discs while taking into account specific parameters for milling acrylic polymers. _ Check milled framework for obvious chips or cracks, use only fault free framework for temporary restorations. _ Ensure that objected is dust-free. _ Use temporary, eugenol-free adhesive cements for installment inside the mouth.
<b>DESIGN</b> _ For crowns, do not fall below the minimum wall thickness of 1mm. _ When designing bridges with one connected bridge unit, the connection cross-section between crown and connected bridge unit may not be less than 12 mm <sup>2</sup> . _ When designing bridges with a range of up to two connected intermediate links, the connection cross-sections between crown and connected bridge units may not be less than 12 mm <sup>2</sup> .
<b>VENEERING</b> _ After milling, TEMP-FIX can be faced using conventional veneering materials. MMA based materials are preferred. _ When veneering composite, TEMP-FIX must always be bonded as described by composite manufacturer.
<b>FINISHING</b> _ For trimming and separating frameworks, use a cross-cut tungsten carbide or separating disc suitable for acrylics.
<b>HINTS</b> Should intolerance to the product be suspected, use it only after an allergy screening test has been performed. Disinfect the milled parts before inserting them in the patient's mouth. Implant hygiene is the patient's responsibility. These instructions for use are based on our current levels of knowledge. The patient will be held fully responsible as regards correct use of the product. The manufacturer is not liable for any unsatisfactory results, given that it has no influence on processing procedures. The dental technician shall inform the dentist as regards the operations to be performed (such as sterilization for example) before inserting the part into the patient's mouth, making sure that the dentist has received all of the information contained in these instructions for use. Any claims for damages shall concern exclusively the commercial value of this product.

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