Harvest VersaGlue™

High performance adhesives that offer exceptional bond strength, instant cure, and extended shelf life.

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INSTRUCTIONS FOR USE – ENGLISH

Intended Use:

For the bonding of gypsum, metals, acrylics, and porcelain as needed for use in various dental laboratory applications.

Technical Data:

Available in 3 levels of viscosity. Details on each are outline below;
VersaGlue PX-4 Thin: Thin viscosity for optimal flow into tight and hard to reach places.

- Set Time: 1-5 seconds
- Applications: gluing articulating systems, repairing gypsums, fixing broken denture teeth

VersaGlue PX-40 Medium: Medium viscosity glue exhibits exceptional gap filling properties.

- Set Time: 10-15 seconds
- Applications: dowel pin placement, gluing articulating systems, repairing gypsums, joining broken dies, bonding bridge segments

VersaGlue PX-400 Thick: High viscosity glue for closing large gaps. Thick and strong.

- Set Time: 30-60 seconds
- Applications: dowel pin placement, blocking out undercuts, filling voids & bubbles, stabilizing abutments, bonding bridge segments

Hazard Warnings:

When using, please use personal protective clothing (dust mask, gloves, protective goggles)

Working Instructions:

Based on the type of bonding application, select appropriate VersaGlue viscosity. Simply remove cap and using the fine tip applicator on the bottle, squeeze out the desired amount of VersaGlue directly to the working area. Let the adhesive set for the appropriate set time as listed above depending on the viscosity. To instant cure VersaGlue, spray on an even layer of Harvest ZBase Accelerator.

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