Eigenschaften / Properties

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CeraFirm - The universal mixing system for ceramics

- For extremely firm ceramic mixes.
- For all HF and LF ceramics.
- Glycol free - no color influence.
- Time saving - no inbetween drying necessary.
- Lowest possible contraction.
- Highest possible edge sharpness after baking.

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CeraFirm Fluid may be used just like any other conventional fluid for mixing. But CeraFirm has a completely different effect and you will be amazed about the firmness and stability of your ceramic mixed with CeraFirm. By adding a few drops of CeraFirm Add before mixing your stack of ceramic powder with CeraFirm Fluid, you will increase the stability even more and get an almost gel-like ceramic mix. Even vibrations and condensing the ceramic with an instrument will not affect your layered structure as it did in the past.

CeraFirm saves you from hassling with in-between drying – now you just need to do this once when you are done with layering. You will find that CeraFirm also allows for a longer working time, the ceramic mix no longer dries out that quickly. CeraFirm does not contain glycol nor glycerin, and therefore has no influence on the color of finished work. The contraction during baking corresponds to the effect you achieve with purified water and the baked ceramic shows enormous edge sharpness. By using of CeraFirm fluid in combination with CeraFirm add, it is possible to process the various ceramic stacks of a restoration in different consistencies. This allows you to layer dentin structures and mamelons with a firmer consistency and using a softer consistency to layer Incisal and Transpa over, not risking to harm the underlying structures created before. Everything is possible – YOU decide what works!

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