



Harvest FixGel™

Temporary Holding Gel
Gel de fixation temporaire

Harvest Dental Products, LLC
905 Columbia St.
Brea, CA 92821 USA
Tel: 1+714-674-7400
Fax: 1+714-674-7402
Em: hello@harvestdental.com
Web: www.harvestdental.com

INSTRUCTIONS FOR USE – ENGLISH

Intended Use:

Temporary Holding Gel for Ceramics, Zirconia und PFM • Firmly holds veneers, copings and crowns on the die.

- 100% organic - burns residue free.
- Slight white color allows visual control.
- No drying time.

Working Instructions:

FixGel is applied straight from the syringe either occlusal onto the die or into the coping. Place your coping on the die and slightly push it on. Due to its' white color you can visually control the even coating of FixGel between die and coping displacing the air between die and coping. Excess material is pressed out cervically and needs to be removed with a paper towel before taking the following steps to layer your ceramic as usual. Carefully release the coping from the cervical margin using an instrument. Cleaning the inside of the coping before baking the ceramic is not necessary, because FixGel burns without residue.

MODE D'EMPLOI – FRANÇAIS

Utilisation prévue :

Gel de fixation temporaire pour les céramiques, la zircone et la PFM • Fixe fermement les facettes, les chapes et les couronnes sur le die.

- 100 % biologique – brûle sans laisser de résidus.
- Couleur légèrement blanche qui favorise le contrôle visuel.
- Aucun temps de séchage.

Instructions d'utilisation :

FixGel s'applique directement à partir de la seringue de manière occlusale sur le die ou dans la chape. Mettez votre chape sur le die et appliquez-la avec une force légère. Grâce à sa couleur blanche, vous pouvez visuellement contrôler l'application uniforme du FixGel entre le die et la chape ainsi que le déplacement de l'air entre le die et la chape. Toute matière excédentaire est évacuée de manière cervicale et doit être enlevée à l'aide d'une serviette en papier avant de procéder aux étapes suivantes habituelles pour disposer la céramique en couches. Libérez soigneusement la chape de la marge cervicale en utilisant un instrument. Il n'est pas nécessaire de nettoyer l'intérieur de la chape avant de cuire la céramique puisque FixGel brûle sans laisser de résidus.