Harvest Poly Cast™

Transparent acrylic for clean burnout.
 Transparenter rückstandslos verbrennbarer Kunststoff
 Material sintético para procesar en la fresadora CNC

Harvest Dental Products, LLC
 905 Columbia St.
 Brea, CA 92821 USA
 Tel: 1+714-674-7400
 Fax: 1+714-674-7402
 Em: hello@harvestdental.com
 Web: www.harvestdental.com

| | |
|---|--|
| INSTRUCTIONS FOR USE – ENGLISH | |
| INTENDED USE Harvest Poly Cast™ _ Can be cast, pressed or over-pressed. _ Check of the fit and function of the laboratory restoration on the stone model and intraorally. _ Not suitable for longer-term intraoral wear (max. 5 minutes). | |
| TECHNICAL DATA Softening point: > 102°C Ignition Point: >102°C | |
| COMPOSITION _ Polymethyl methacrylate (PMMA) | |
| SHADE Colorless or colored. | |
| HAZARD WARNINGS: Dust is created during milling of POLY CAST blanks that can cause irritation of the eyes, skin and airways. Therefore always ensure that the extractor on your milling machine is functioning perfectly. | |
| STORAGE No special measured required. | |
| SHELF LIFE Up to 10 years from date of manufacture. | |
| WORKING INSTRUCTIONS Follow the manufacturer's instructions for materials in which the substructure is cast or pressed. To prevent the investment material from cracking, it is advisable to cover the framework with a thin layer of wax before investing. The PMMA framework must be cleaned and disinfected according to hygiene guidelines before placement in the mouth. | |
| GUARANTEE Application recommendations, no matter whether verbally, in writing, or by means of practical instructions are based on our experience and tests and are thus only a recommendation. Our products are subject to continuous further development. We reserve the right to alter the products in terms of design or composition. | |

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