Harvest Temp Esthetic™

TEMP, TOOTH SHADES
 TEMP, TEINTES DE DENTS
 TEMP, ZAHNFARBEN
 TEMP, TONALIDADES DEL DIENTE

Harvest Dental Products, LLC
 905 Columbia St.
 Brea, CA 92821 USA
 Tel: 1+714-674-7400
 Fax: 1+714-674-7400
 Em: hello@harvestdental.com
 Web: www.harvestdental.com

INSTRUCTIONS FOR USE – ENGLISH
<p>INTENDED USE The Harvest Temp Esthetic PMMA Block is used to fabricate crowns, bridges and structures for implant supported provisional removable denture and appliance prosthetics.</p> <ul style="list-style-type: none"> - Provisional anterior and posterior crowns & bridges - Implant and abutment supported prosthetics. - Partial, complete and hybrid denture prosthetics (base and teeth) <p>This device is intended for all patient populations with no known subpopulations to which the device cannot be used.</p>
<p>RESISTANCE E-MODULE: > 2.200 MPa BENDING STRENGTH: > 100 MPa</p>
<p>COMPOSITION _Polymethyl methacrylate (PMMA) and ester-based cross- linked polymers. _Color pigments.</p>
<p>SHADE BL2, A1, A2, A3, A3,5, A4, B1, B2, C1, C2, C3, D2, D3</p>
<p>HAZARD WARNINGS: Dust is created during milling of TEMP ESTHETIC blanks that can cause irritation of the eyes, skin and airways. Therefore always ensure that the extractor on your milling machine is functioning perfectly.</p>
<p>STORAGE / SHELF LIFE / DISPOSAL 10 Years PMMA can either be collected for recycle or disposed of in general waste. No special storage measure requirements.</p>
<p>WORKING INSTRUCTIONS _Mill resin discs while taking into account specific parameters for milling acrylic polymers. _Check milled framework for obvious chips or cracks, use only fault free framework for temporary restorations. _Ensure that objected is dust-free. _Use temporary, eugenol-free adhesive cements for installment inside the mouth.</p> <p>DESIGN _For crowns, do not fall below the minimum wall thickness of 1mm. _When designing bridges with one connected bridge unit, the connection cross-section between crown and connected bridge unit may not be less than 12 mm². _When designing bridges with a range of up to two connected intermediate links, the connection cross-sections between crown and connected bridge units may not be less than 12 mm².</p> <p>VENEERING _After milling, TEMP ESTHETIC can be faced using conventional veneering materials. MMA based materials are preferred. _When veneering composite, TEMP ESTHETIC must always be bonded as described by composite manufacturer.</p> <p>FINISHING _For trimming and separating frameworks, use a cross-cut tungsten carbide or separating disc suitable for acrylics.</p> <p>HINTS Should intolerance to the product be suspected, use it only after an allergy screening test has been performed. Disinfect the milled parts before inserting them in the patient's mouth. Implant hygiene is the patient's responsibility. These instructions for use are based on our current levels of knowledge. The patient will be held fully responsible as regards correct use of the product. The manufacturer is not liable for any unsatisfactory results, given that it has no influence on processing procedures. The dental technician shall inform the dentist as regards the operations to be performed (such as sterilization for example) before inserting the part into the patient's mouth, making sure that the dentist has received all of the information contained in these instructions for use. Any claims for damages shall concern exclusively the commercial value of this product.</p>
<p>TYPICAL MECHANICAL PROPERTIES OF PMMA BLOCKS Modulus OF Elasticity: > 2,200 MPa Bending Strength: > 100 MPa Toughness: > 10 KJ / m2 (Izod): > 1.3 KJ / m2 Tensile Strength: > 75 MPa Vickers Hardness HV (0,05/10): > 135 Absorption of Water according to ISO 10477: < 20 µg/mm3 Water solubility: <0.8 µg/mm3 Residual Monomer Content (24 h at 37°C and 30 min in air): < 0.7 % Burning residual (2 h 500°C) (Clear) Bond Strength > 5Mpa</p>

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