



Tray by Harvest

Ceramic Mixing Palette

Contents:

- 1 Tray aluminum body
- 1 Glass mixing pad
- 1 Stand for lid
- 1 Water dispenser bottle
- 1 White dry erase pen
- 1 Smile Maker poster

Harvest Dental Products, LLC
 905 Columbia St. Brea, CA 92821
 714.674.7400
 www.harvestdental.com
 hello@harvestdental.com



Instructions for use

Remove Tray aluminum body and set on your bench. Set Tray aluminum lid in provided 'stand' along with dry erase Pen.



1 Fill bottle with water and fill reservoir track starting at the top.



2 Immerse wetting strip into reservoir to full water absorption.



3 Feed wetting Strip through Slot provided in glass pad.



4 Extend wetting Strip horizontally across glass and feed through slot.



5 Place glass into aluminum base ensuring strips rest in reservoir.



6 Place powders against strip for optimal hydration.



7 Use lid as dry erase board for whatever you wish.



8 Enjoy your experience with Tray. :-)