

A Better Way to the Perfect Restoration.

TruePrep. Stump shade management from the leader in die preparation.



Overview

TruePrep Natural Shade Die Spacers simulate the shade of the prepared tooth structure in the oral environment right on the master model. This serves as the most optimum basis for true-to-nature shade reproduction of the oral environment of the patient, thus becoming an essential foundation for the highly esthetic restoration to be produced. TruePrep provides accurate die cement space simulation on the master die, while eliminating the need to fabricate time consuming, light cure composite dies.

Once the dentist has prescribed an IPS Natural Die Material shade (i.e. ND 1), select the TruePrep HD shade which directly corresponds to same number (i.e. HD 1).

Technical Info

Micron per coat: 12µ
 Unit of measure: 7 ml
 Shade Guide basis: IPS e.max® Natural Die Material®

Instructions for Use:

1. Proceed with model preparation using a light colored die stone (White, Ivory, or Buff). **NOTE: Shades may slightly differ based on color and porosity of dental stone.**
2. Choose the prescribed IPS ND shade number by selecting the corresponding TruePrep HD shade with the same number.
3. Apply TWO coats (24µ) of TruePrep Spacer starting at the occlusal surface, coming down to ½ mm above the margin line.
4. Let dry for 15 seconds. No additional sealant is required. Material can be thinned by adding Thinner (Item 7510, sold separately).

Stump Shade Conversion Chart				
TruePrep Item #	TruePrep Stump Spacers	IPS Natural Die Material® Stump Shades	IPS Empress® Stump Shades	VITA® Lumin® Shades
7501	HD 1	ND 1	-	A1 / B1
7502	HD 2	ND 2	ST 9	A2 / A3 / B2
7503	HD 3	ND 3	ST 1	B3 / B4
7504	HD 4	ND 4	ST 2	-
7505	HD 5	ND 5	ST 3 / ST 4	-
7506	HD 6	ND 6	ST 6	-
7507	HD 7	ND 7	ST 7 / ST 8	A3.5 / A4 / C1 / C2 / C3 / D2 / D3 / D4
7508	HD 8	ND 8	ST 5	C4
7509	HD 9	ND 9	-	-

North America
 Harvest Dental Products, LLC
 905 Columbia St.
 Brea, CA 92821 USA
 Ph: 714-674-7400 Fx: 714-674-7402
 www.harvestdental.com

Outside of North America
 HPdent, GmbH
 Anneliese-Bilger-Platz 1
 78244 Gottmadingen, Germany
 Ph: +49 7731 38 11 044 Fx: +49 7731 31 97 123
 www.hp-dent.com

