Harvest LC DuraPattern™

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INSTRUCTIONS FOR USE – ENGLISH

Intended Use:

For efficient modeling for scanning or casting of the following indications;

- Implant jigs, abutments, bars, supra-structures, and transfer guides

For efficient modeling for scanning, pressing, or casting of the following indications;

- Crowns, bridges, inlays and onlays
- Attachments

Technical Data:

Can be light cured using most conventional Light Curing devices on the market today.

Average curing time based on type of light cure unit are listed below;

- UV/UV-A lights: 3.5 to 5 minutes
- Halogen Lights: 2 minutes
- Strobo Lights: 1.5 minutes

Hazard Warnings:

Non-hazardous. When processing, please use personal protective clothing (dust mask, protective goggles)

Storage:

Store in a dry place in its original packaging (<25 deg. C). Keep away from direct sunlight.

Working Instructions:

Coat dental stone working surface with MS3 Master Die Separator prior to application. Use LC Dura Pattern Flow syringe for easy application when more control is need for thin and delicate modeling. For larger surface areas, use LC Dura Pattern Paste directly out of the container with a wax knife or spatula. These tools along with the technician's fingers can easily mold the material in place prior to Light Curing. Close cap on syringe and/or jar immediately after use of the LC Dura Pattern material is complete.

Simply Light Cure the material until it has cured and then press, cast, or scan depending on your desired indication. You can reference the Technical Data above for estimated cure times based on type of light cure unit used.

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Harvest Dental Products, LLC