

INSTRUCTIONS FOR USE – English

WORKING INSTRUCTIONS

- It's advisable to start a recycling cycle as soon as the liquid in action is white and opaque, but before it starts to get jelly.
- Instead of distilling the liquid like the machine Sidewinder does, we "polymerize" the resin inside the bowls/containers once the liquid is no longer effective enough.
- Possibility A: put the bowl/container near a window with daylight for a timeframe between 1 and 4 hours depending on the quantity of solved resin.
- Possibility B: put the bowl/container in a UV light curing machine – timeframe between 1 and 3 hours depending on the quantity of solved resin.

INSTRUCCIONES DE USO - Español

INSTRUCCIONES DE TRABAJO

- Es recomendable iniciar un ciclo de reciclaje tan pronto como el líquido en acción sea blanco y opaco, pero antes de que comience a gelatinizarse.
- En lugar de destilar el líquido como lo hace la máquina Sidewinder, "polimerizamos" la resina dentro de los tazones/recipientes una vez que el líquido ya no es lo suficientemente efectivo.
- Posibilidad A: colocar el bol/recipiente cerca de una ventana con luz natural durante un tiempo de 1 a 4 horas dependiendo de la cantidad de resina disuelta.
- Posibilidad B: poner el bol/recipiente en una máquina de fotopolimerización UV – tiempo entre 1 y 3 horas dependiendo de la cantidad de resina disuelta.