



Model Sep

Separating agent for gypsum dental models.

Agente separador para modelos dentales de yeso.

Harvest Dental Products, LLC
905 Columbia St. Brea, CA 92821 USA
Tel: 1+714-674-7400
Fax: 1+714-674-7402
Email: hello@harvestdental.com
Web: www.harvestdental.com

INSTRUCTIONS FOR USE – English

INTENDED USE

Harvest Model Sep is used in the dental industry to cleanly separate gypsum dental models.

WORKING INSTRUCTIONS

Direct Application

- Apply by brushing or spraying the model with Model Sep, making sure to cover all surfaces with an even coat.
- Model Sep will absorb into the gypsum model, in some cases two coats may be needed.
- Pour stone or plaster immediately against the gypsum model
- Blow or shake away excess solution

GUARANTEE

Application guidelines are only a recommendation. We reserve the right to alter the products in terms of design or composition.

INSTRUCCIONES DE USO - Español

USO PREVISTO

Harvest Model Sep es utilizado en la industria dental, para la separación limpia de modelos de yeso.

INSTRUCCIONES DE TRABAJO

Aplicación directa

- Aplique directamente Model Sep con Pincel o Spray, Asegurando que todas las partes del modelo fueron cubiertas con una delgada capa.
- El yeso Absorberá el Model Sep. Dependiendo el tipo de yeso una segunda capa será necesaria.
- Vierta el yeso o el Plaster inmediatamente contra el modelo de yeso.
- Sople o sacuda el exceso de solución.

GARANTÍA

Las pautas de aplicación son solo una recomendación. Nos reservamos el derecho de alterar los productos en términos de diseño o composición.